

SleepWeaver 3D FAQs

1. Is the SleepWeaver 3D a specialty mask?

The SleepWeaver 3D Nasal Mask is intended to provide an interface for Continuous Positive Airway Pressure (CPAP) or Bi-Level therapy. This mask is intended for single-patient reuse in the home and single-patient reuse in the hospital / institutional environment. This mask is to be used on patients greater than 66 lbs (30 kg).

2. Does the SleepWeaver 3D Mask come in different sizes?

The soft cloth portion of the mask is only available in one size and the headgear is available in regular and large. The most accurate measurement is directly under the jawline. If you measure 16 inches (41cm) or above, then you should consider the large size headgear. If you measure 16.5 inches (42 cm) or less, then you should consider the regular size headgear. The SleepWeaver 3D is intended to be used on individuals who are greater than 66 lbs (30 kg).

3. What should I do if I am experiencing leaks with my SleepWeaver 3D?

You should make sure your measurements are accurate. If your measurement falls between the two headgear sizes, then you should try each size headgear to see which one provides the best fit. You can also review the fit tip video online at <http://circadiance.com/3dvideos/>.

4. Does the mask come in different colors?

No. The SleepWeaver 3D is available only in blue.

5. From what material is the mask made?

The cloth is the same material used to make winter ski clothing. A portion of the mask that contacts the skin contains polyester, nylon, and elastane. The cloth material is “breathable” meaning that the material wicks moisture and heat away from the user’s skin. The headgear consists of polyurethane foam, nylon, and lycra.

6. Does the cloth mask contain latex?

No. The mask is not made from natural rubber latex.

7. Do you offer a warranty?

Yes. Circadiance warrants that your SleepWeaver 3D mask and headgear shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

8. Do you have a mask guarantee?

If you, the patient, purchased one of our SleepWeaver products (SleepWeaver Advance, SleepWeaver Advance Small, SleepWeaver Advance Pediatric, SleepWeaver Anew, SleepWeaver 3D or SleepWeaver Élan) through your Home Care Provider or an Internet Reseller and do not like the product, you can return it to the location where it was purchased within 30 days of initial fitting or 30 days of receiving the product.

9. Can I replace the swivel on SleepWeaver 3D?

The swivel is permanently attached to the mask and cannot be replaced.

10. Will insurance cover the cost of a mask?

In the U.S., the SleepWeaver 3D is covered under CMS codes A7034 & A7035 for Nasal Mask and Headgear.

11. How long will it take to become comfortable with the SleepWeaver 3D?

Most users report that after the first night using our SleepWeaver product it is the most comfortable mask they have ever tried! However, it can sometimes take longer than one night to become accustomed to new equipment, especially if you are new to CPAP and sleep therapy. Please remember that the SleepWeaver 3D is a loose fitting PAP mask. DO NOT overtighten the headgear because it could cause you to experience mask leaks. Follow the instructions provided by your medical equipment supplier. We also encourage you to view the [SleepWeaver 3D](#) videos, read the instructions and call us for support if necessary (888-825-9640). Once you are comfortable with your new SleepWeaver 3D mask, you too will experience the restful sleep you have been missing.

12. In what position can the tubing be run when using a SleepWeaver product?

With SleepWeaver 3D, it is recommended you place the hose down towards the user’s chest, or off to either side. If you wish to direct the tubing over their head, i.e. behind the headboard of the bed, ensure that a sufficient length of tube is available so that no major kinks are in the tubing. Some PAP users connect 2 lengths of tubing to provide a longer hose which allows greater freedom of movement. Circadiance also sells the following accessories to assist in positioning the hose to your preference: Feather Weight Tube, Tether Straps, and Hose Clips.

13. Will SleepWeaver 3D work with my existing CPAP or Bi-Level machine?

SleepWeaver products are designed to work with standard CPAP and Bi-Level machines.

SleepWeaver 3D FAQs

14. Will the mask attach to any brand of machine?

The fitting on the SleepWeaver 3D fits the 22mm ISO standard port which is used on most PAP machines.

15. Can I use the SleepWeaver with a humidifier?

Yes.

16. Can I use Supplemental Oxygen?

Yes. The mask can be used with machines that have that option. Refer to your machine's user manual for more information.

17. How long will SleepWeaver 3D last?

SleepWeaver 3D is warranted for 90 days. Many of our users report wearing it for a longer period of time.

18. How do I clean the SleepWeaver 3D Mask?

The SleepWeaver 3D should be cleaned by hand washing with a mild, nonabrasive soap in warm water every other day. Never use hot water. Repeat washing until visibly clean. Hand wash the headgear weekly. Rinse well with quality drinking water and allow to air dry out of direct sunlight.

19. Does the plastic stay need to be removed from the headgear for cleaning?

No. It is recommended that you DO NOT remove the plastic stay from the headgear.

20. Does SleepWeaver 3D have an exhalation valve?

The exhalation "valve" on the SleepWeaver 3D is a series of very small holes that gently and quietly exhaust the exhaled air from the mask. These holes provide quiet operation without creating a jet of air that make a lot of noise. The leak rate provided by the exhalation holes is comparable to competitive CPAP masks.

21. Can the exhalation valve get plugged or clogged?

As long as the mask is properly maintained and washed, the exhalation valve will provide consistent exhaust performance.

22. Can I sleep on my stomach while wearing the SleepWeaver 3D?

Yes. The air pressure in the mask will keep the mask inflated without pressure points. The mask continues to provide CO₂ exhalation as long as a portion of the exhalation holes are exposed.

23. Can I read or watch TV while wearing the mask?

There is nothing in the field of view so users can read or watch TV and even wear eyeglasses with the mask on.

24. Can I travel with SleepWeaver 3D?

Yes. The SleepWeaver (3D, Advance, Élan, Anew and Advance Pediatric) product lines are the smallest and lightest masks for travel available. They fold into a very small size when not inflated so they will fit easily into any carrying case you would use for traveling.

25. Does the SleepWeaver 3D have FDA Clearance and a CE mark?

Yes. The SleepWeaver 3D has been FDA cleared and does have a CE mark.

26. Can I use the SleepWeaver 3D Mask with a broken nose or deviated septum?

We have had a number of users with anatomic issues around the nose report using the SleepWeaver with success. The pliable nature of the SleepWeaver soft cloth CPAP masks allows it to conform to the user's facial anatomy even if it is not symmetrical.

27. Does the SleepWeaver 3D address the problems of people who have skin reactions to plastic masks?

The SleepWeaver 3D is designed to be gentle on the skin. We have had a number of user's report that their skin irritation problems disappeared after switching to a SleepWeaver mask.

28. Do you sell through Home Care Companies?

Yes. We sell through all Home Care Companies. If your company does not carry the SleepWeaver 3D, please have them contact us so that we can set them up as a distributor. Alternatively, you can contact us and give us the name of your Home Care Company so that we can have our sales representative call on them.

29. Do you sell through any National DME chains?

Yes. We sell through national chains like Lincare, Apria, American Home Patient, AeroCare, and many more. Please visit us online at <http://www.circadiance.com/order/dealer-locator/> to find a dealer near you.

30. Do you have a sales representative for this product?

Circadiance has a sales force that calls on Home Care Companies. Please contact us with your name, company and address and we will have our local representative contact you. [Click Here to Submit Your Information](#)

31. Do you have a Sleep Lab program?

SleepWeaver 3D FAQs

Yes. Please contact us for the details of our Sleep Lab Program. [Click Here to Submit Your Information](#)

32. Do you sell your mask outside of the U.S.?


Yes. We can sell our products in countries for which we have regulatory approval. [Click here to view our international webpage.](#)

33. Can the headgear for each SleepWeaver mask be used with other SleepWeaver products?

No. Each SleepWeaver headgear is specific to its mask and it cannot be interchanged.

34. What is the recommended pressure?

We have tested SleepWeaver 3D at pressures from 4 cm H₂O up to 20 cm H₂O on CPAP and Bi-Level systems.

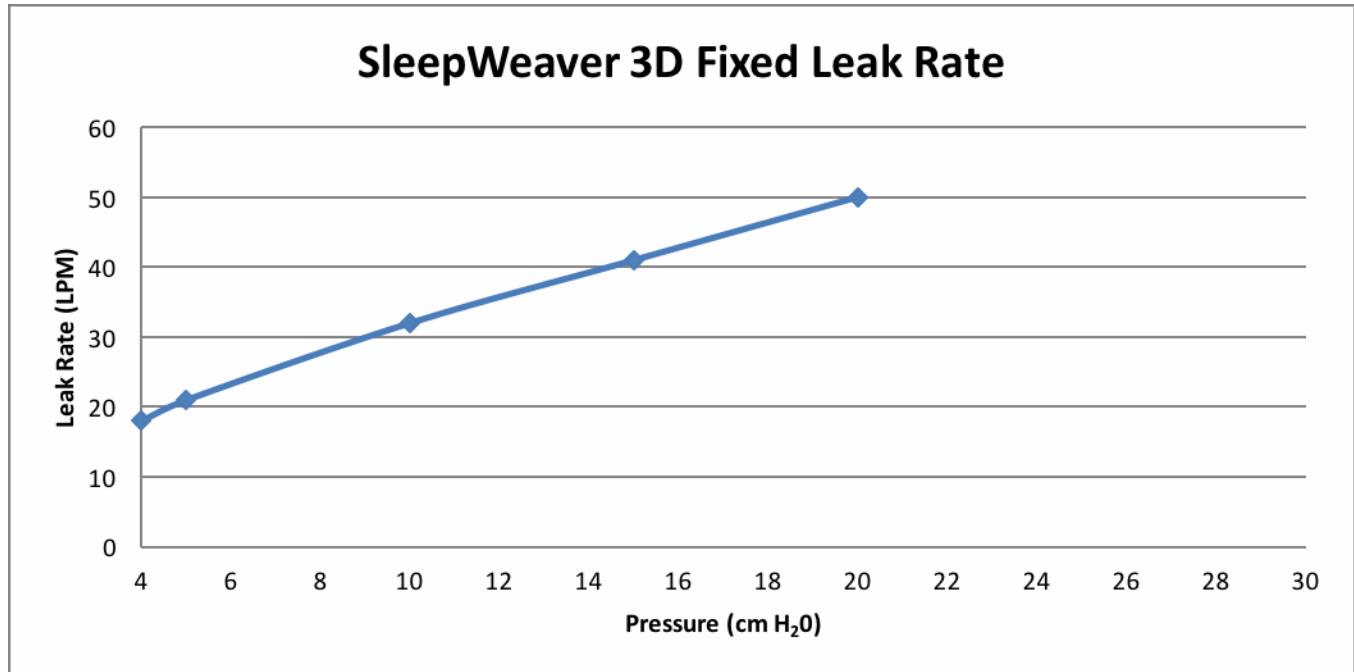

SleepWeaver 3D Fixed Leak Rate

Pressure (cm H ₂ O)	4	5	10	15	20
Fixed Leak Rate (LPM)	18	21	32	41	50