


**PLEASE READ BEFORE**  
**PRINTING**

**Print page 2 and 3 of this document at 100% to ensure you are sizing yourself properly!**

**To double check your printing accuracy, compare the printed measure guides with a standard size ruler.**

**A. Measure width of nose:**

1"  
1.25"  
1.5"

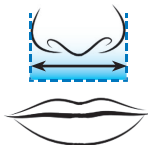

**Nasal Mask Measurement**


- Using the ruler above measure the width of your nose.
- Using your nose measurement, find your mask size in the chart below.

Nose Size	*Advance	Advance Small	Elan®
<b>Small</b>		●	●
<b>Regular</b>	●		●
<b>Large</b>	●		●

\*SleepWeaver Advance is one size fits most. If your nose measurement falls under 1.25" try SleepWeaver Advance Small first.

**SleepWeaver 3D Neck Sizing Guide**

Not actual size  
Use cloth tape measure

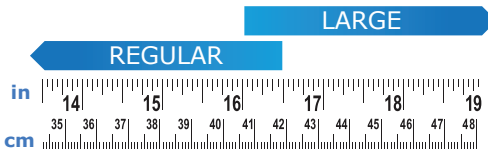

**NOTE: If your measurement falls between sizes on ANY mask, choose the SMALLER mask size.**


**A. Measure width of nose:**

1"  
1.25"  
1.5"



**Full Face Mask Measurements**


- Find measurement A. using top ruler.
- Find measurement B. using side ruler.
- Using both measurements, find your mask size in the chart below.

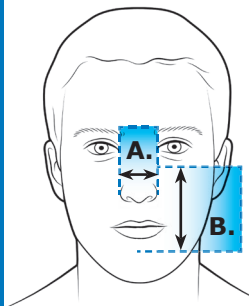

Anew® Size	A. Measurement	B. Measurement
<b>Small</b>	1" to 1.25"	3.25" to 3.5"
<b>Regular</b>	1.25" to 1.5"	3.5" to 3.75"
<b>Large</b>	1.5" & Up	3.75" & Up


**B. Measure from bridge of nose to center of chin:**

3.25"  
3.5"  
3.75"

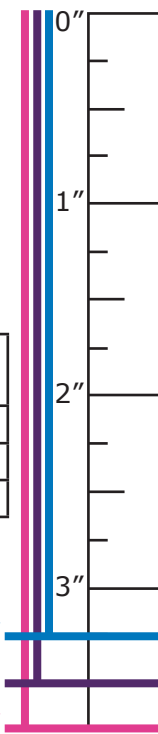


**A. Measure width of nose:**

25mm  
32mm  
38mm

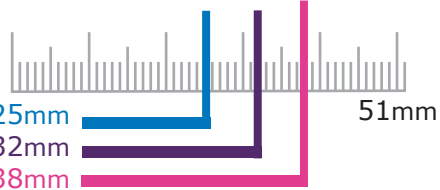

51mm

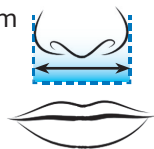

### Nasal Mask Measurement


- Using the ruler above measure the width of your nose.
- Using your nose measurement, find your mask size in the chart below.

Nose Size	*Advance	Advance Small	Elan®
<b>Small</b>		●	●
<b>Regular</b>	●		●
<b>Large</b>	●		●

\*SleepWeaver Advance is one size fits most. If your nose measurement falls under 1.25" try SleepWeaver Advance Small first.

### SleepWeaver 3D Neck Sizing Guide


Not actual size  
Use cloth tape measure

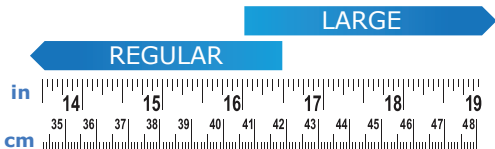

**NOTE: If your measurement falls between sizes on ANY mask, choose the SMALLER mask size.**

**A. Measure width of nose:**


25mm  
32mm  
38mm

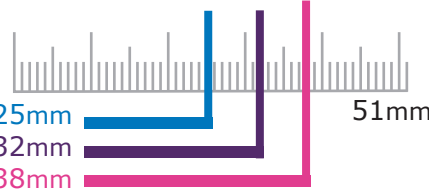

51mm

### Full Face Mask Measurements

- Find measurement A. using top ruler.
- Find measurement B. using side ruler.
- Using both measurements, find your mask size in the chart below.

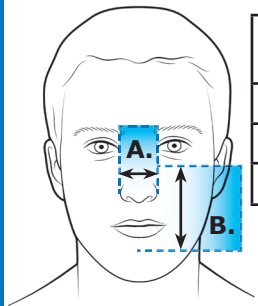

Anew® Size	A. Measurement	B. Measurement
<b>Small</b>	<b>25 to 32mm</b>	<b>83 to 89mm</b>
<b>Regular</b>	<b>32 to 38mm</b>	<b>89 to 95mm</b>
<b>Large</b>	<b>38mm &amp; Up</b>	<b>95mm &amp; Up</b>

**B. Measure from bridge of nose to center of chin:**

83mm  
89mm  
95mm

